

Pharmacy Bag Holders

Installation Instruction Part Number

- 21061521-01

Product Numbers


- PHBHxx

Hardware Pack


- N/A

Required Tools

- N/A

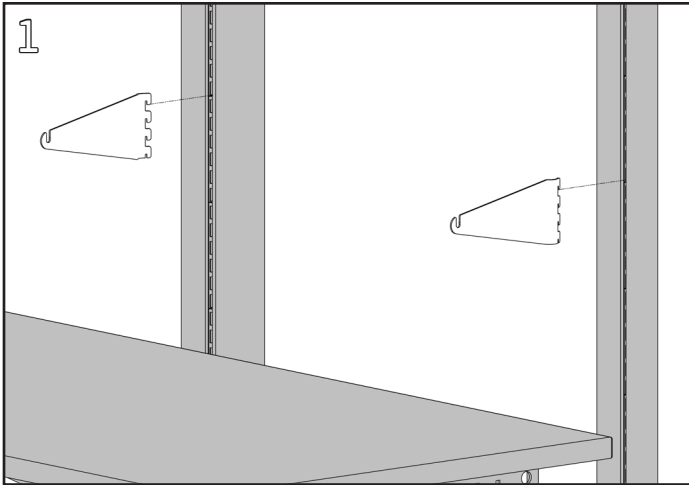

Step 1: With notched edge of bracket up, engage the left-hand bracket into the left side of the frame. Repeat with right-hand bracket on the right side. Ensure the brackets are at the same height and are firmly seated in the hanger frame slots.

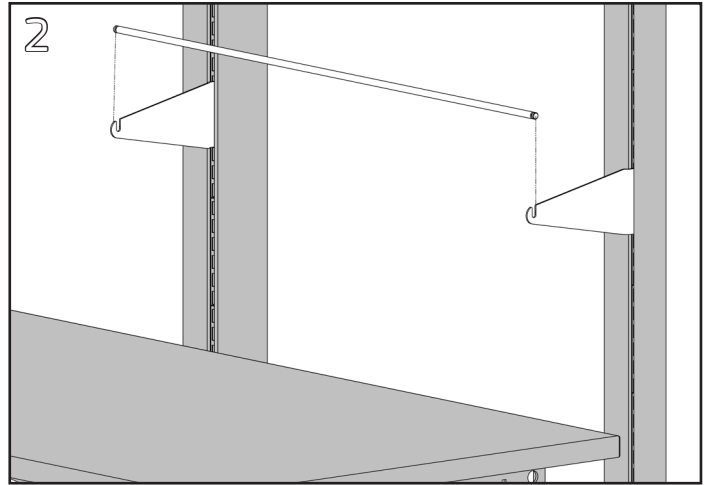

Step 2: Insert the narrow portions of the rod into the notches in each bracket and press down until the rod snaps into place at the bottom of the notch.